


Invia® Liberty™

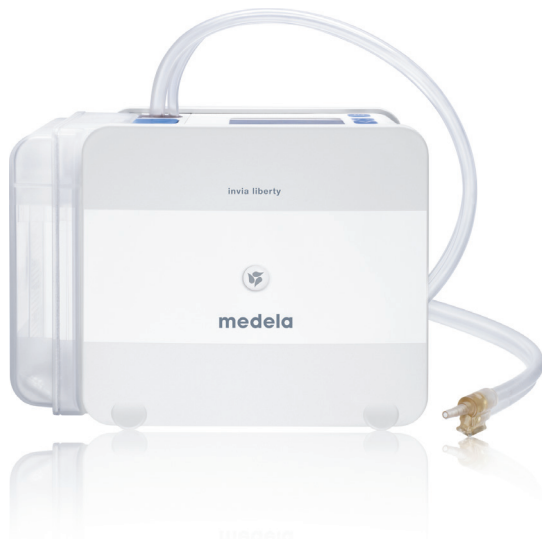

PROVIDING TRUE NEGATIVE
PRESSURE WOUND THERAPY



Invia® Liberty™

Providing Negative Pressure Wound Therapy, Mobility and Flexibility

Negative Pressure Wound Therapy is a preferred choice of care for treating acute and chronic wounds in hospitals, clinics, long-term care and home care settings.


Patients appreciate the ease-of-use, quiet operation and compact size of the Invia Liberty. The device can be carried in a discreet bag, facilitating mobility, and the docking station allows for easy charging.

Invia Liberty's innovative design delivers excellent functionality with its simple set-up and easy operation.

New

Firmware 1.11

- Slower vacuum build-up time
- New air leak indicator
- User friendly mute alarm button
- Reduced time to detect and notify blockage in canister tubing
- Nighttime mode for LED screen

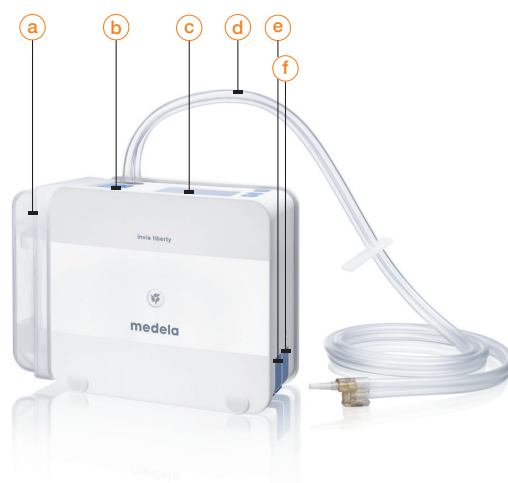

- (a) Disposable canister 300 ml and 800 ml options with solidifier
- (b) One-touch canister release button
- (c) Easy to read LCD screen for intuitive operation and graphic capability
- (d) Double lumen tubing (PVC) with integrated protection filter and tube clamp
- (e) Covered, waterspray safe power adapter port
- (f) Covered, waterspray safe USB port for software updating and therapy history upload to a PC

Indications for Use

Invia Liberty is indicated for use in:

- Diabetic / neuropathic ulcers
- Pressure ulcers
- Chronic wounds
- Acute wounds
- Dehisced wounds

Contraindications


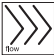
Do not use Invia Liberty for patients with any of the following conditions:

- Malignancy of the wound
- Untreated osteomyelitis
- Unmanged malnutrition
- Non-enteric fistula
- Unexplored fistula
- Necrotic tissue with eschar present


- Do not place Invia Wound Therapy Dressing over exposed blood vessels or organs.
- Do not place directly over anastomose or sutured blood vessels

Medela offers healthcare professionals 50 years of expertise in medical vacuum technology. Our Swiss development and manufacturing process yields mechanical precision to guarantee excellent performance.

Technical data

| | |
|---|-----------------------|
|  | - 60 mmHg to 200 mmHg |
|  | Low Flow 5 L/min. |

| | |
|---|-----------------|
|  | > 12 Hours |
|  | 1 kg 2.2 lbs |
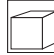
| | |
|---|---|
| Modes: | Constant and Intermittent |
| Tubing: | Double Lumen Tubing, Y-Connector |
| Canister: | 300 and 800 ml |
|  | H x W x D 95 x 170 x 150 mm 3.74 x 6.69 x 5.91 inch |
| 2 year warranty | |

Medical Vacuum Technology
for Healthcare Professionals

Please contact us or your local Medela representative for details.

Federal law restricts this device to sale or rental by or on the order of a physician.

Medela and Invia are registered trademarks of Medela Holding AG. Liberty is a trademark of Medela, Inc.

 Medela AG
Lättichstrasse 4b
6341 Baar / Switzerland
www.medela.com
Medela, Inc.
1101 Corporate Drive
McHenry, IL 60050
USA
Phone +1 877 735 1626
Fax +1 815 307 8942
suction@medela.com
www.medelasuction.com

Local contact: